

# Directions for use

## ***BRS – Rescue Sack***



<b>Contents</b>	<b>Page</b>
Product description	2-3
Purpose	4
Application	4
Cleaning	4
Storage	4
Reusability	4
Helicopterlifting	4
Producer	4

# Product description

Optimised for use with vacuum mattress



Horizontal lifting straps, 4 pcs on each side, colour black are adjustable in 2 positions, 1 pc on each side, colour yellow is fully adjustable to achieve Horizontal position of patient. All straps sewn onto and inseparable from the BRS.

Patient straps with metal buckles, 5 pcs, sewn onto and inseparable from the BRS.



Patient restraint straps and horizontal lifting straps are stowed in 5 pockets on each side.

Plated-through holes for free drainage of water on the underside.

Antiskid underside to prevent slipping on snow.

Anti-rotation rings



Zipper & hook & loop for easy access to the patient

Pocket for storage of vacuum pump.



Blue straps for securing of RedVac vacuum mattress



Label with Serial number, manufacturing date, CE-marking



Replaceable foam insert for head protection



**BRS03**  
**BRS04**

Color: Red

Color: Green

Carry Handles: 10

Patient Restraint Straps: 5

Helicopter Lifting Straps: 10

Width Head/Middle/Foot: 40cm/70cm/45cm

Length: 220cm

Weight: 4kg



**TBRS03**  
**TBRS04**

Color: Red

Color: Green

Carry Bag with Ruck-Sack straps

Width - Length - Height: 44cm/80cm/33cm

Weight: 1kg

## 1. Purpose :

The BRS Rescue Sack is used to protect patients against bad and cold weather during transportation from place of accident to a protected area. For optimal comfort and safety of the patient it is recommended to use the BRS with a vacuum mattress. The applications are limited to preclinical assistance and the first phase of clinical support. It is equipped with horizontal lifting straps and can be lifted with a helicopter.

## 2. Application:

The BRS Rescue Sack is a medical product which may be applied only by medical specialist personnel instructed and trained regularly on the product. The kind of application is in accordance with the current, generally recognised medical guidelines and recommendations of the specialised societies which are concerned. Before usage the horizontal lifting straps must be carefully inspected and found free of any damage. If any damage is found the BRS rescue Sack may not be lifted horizontally.

## 3. Cleaning:

The BRS Rescue Sack can be cleaned with luke warm water and a detergent.

## 4. Storage:

The shall be stored in a dry area free of sunlight. Prolonged exposure to sunlight may damage the sack and the lifting and patient restraint straps.

## 5. Reusability:

The product may be used again if the material is undamaged and its function preserved. Prior to reuse the user must check the proper condition of the product. Only products in perfect technical and hygienic condition may be reused.

## 5. Helicopterlifting:

Helicopter Lifting Permission issued by the German Military:  
D-Luftfahrtverträglichkeitsprüfung: 6545-2005-02 issued 25.11.2009

## 6. Producer :



**KOHLBRAT & BUNZ GmbH**  
Loretostrasse 4-8  
5550 Radstadt  
Austria  
phone: +43-6452-7193-0  
fax: +43-6452-7193-51  
office@kohlbrat-bunz.com  
www.RedVac.com

**Quality Management System according to EN ISO 9001:2008 & EN ISO 13485:2003  
certified by TÜV Austria**