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*The **U**niversal **T**ransporting Device from Kohlbrat & Bunz*

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Operation and Maintenance Instructions

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
Table of Contents

The individual Parts.....	2
Technical Data	3
Operational Applications of UT 2000	3
Utilization as a Back-Carrier Device	5
Utilization as Stretcher or Load Carrier	5
Utilization as a cot or Emergency Bed	6
Utilization as a Sled	7
Uncoupling of 2 UT 2000	8
Helicopter Rescue	9
PRO Helicopter Rescue	10
PRO Helicopter Rescue Adjustabl	11
Helicopter Rescue and Precipice Recovery	12
Main trance and Cleaning.....	13
Accessories: Bracket, Float and AR System	14
Accessories: Wheel Systems.....	15
Accessories: Ackja System	16
Accessories: Head Protection and Pads	17
Maintenance Notice	18
Flight Manual.....	21

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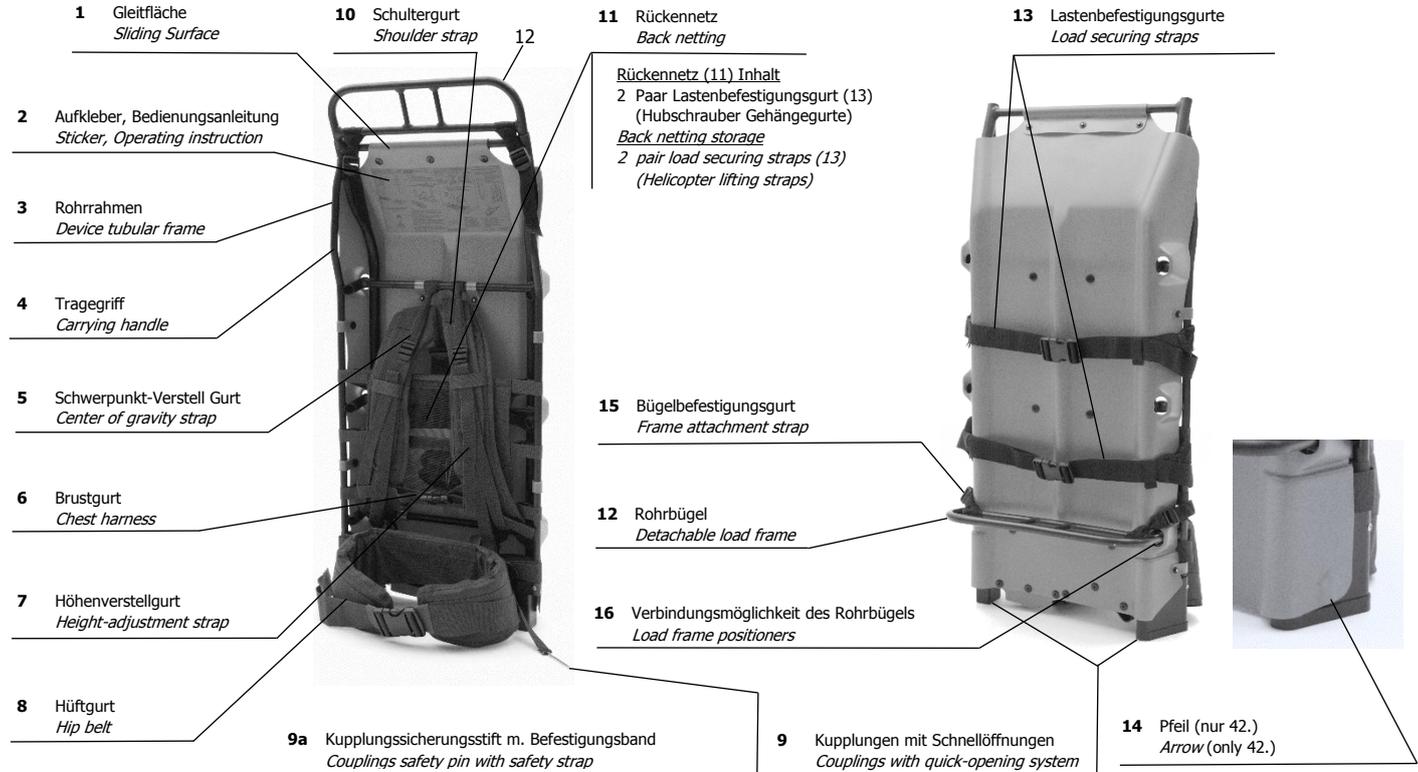Technical Data:**Use as a back carrier:**

without detachable frame:		with detachable frame:
length:	940 mm	1040 mm
height:	120 mm	120 mm
width:	440 mm	440 mm

Empty weight 42. = 4.10 kg / 46. = 4.20 kg

Maximum permissible load: 50 kg.

Temperature resistance from -50 ° C to + 100 ° C

Utilization as sled in snow/towing sled, load carrier, rescue unit, first-aid stretcher, emergency cot and helicopter rescue:

	with/without detachable frame	42. /	46.
length:	1810/2000 mm	empty weight:	8,20 kg/8,40 kg
width:	440 mm	maximum permissible load:	
height:	120/220 mm	150 kg, 1 person	

OPERATIONAL APPLICATIONS OF THE UT 2000

The UT 2000 can be employed for the following assignments either as a single device or when two devices are joined together:

1. Back-carrying device for transporting loads of all kinds of equipment.
2. Cargo Carrier for loads up to 150 kg (2 UT 2000 coupled together)
3. Stretcher for transporting an injured person (2 UT 2000 coupled together)
4. Sled in snow for the transportation of loads (1 or 2 UT 2000) or an injured person (2UT 2000s coupled together)
5. Precipice Recovery (2 UT 2000 couples together)
6. Helicopter Suspension for loads up to 150 kg, respectively for an injured person (2 UT 2000 couples together)
7. Cot, Emergency Bed (2 UT 2000 coupled together)

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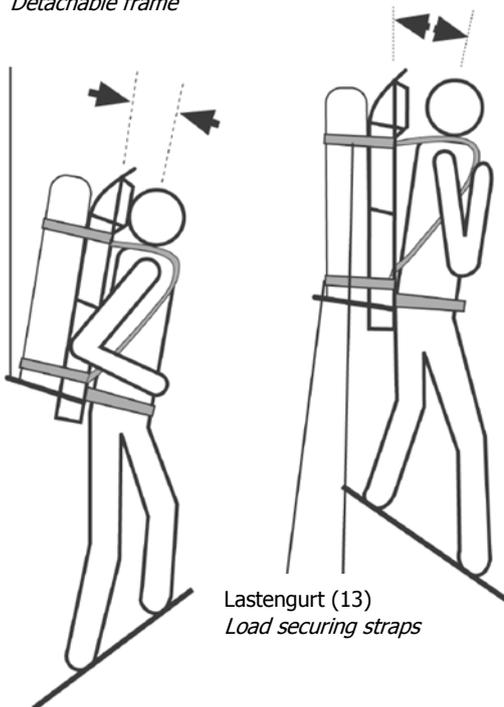
Position the centre of gravity by adjusting the straps

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

If using as a pull able sled, load carrier/first-aid stretcher, the two couplings must be secured via the securing pin.

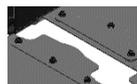Konnektor
connector

Version 42.
Fahrtrichtung in Richtung des Pfeiles.
Direction of travel in the direction of the arrow.

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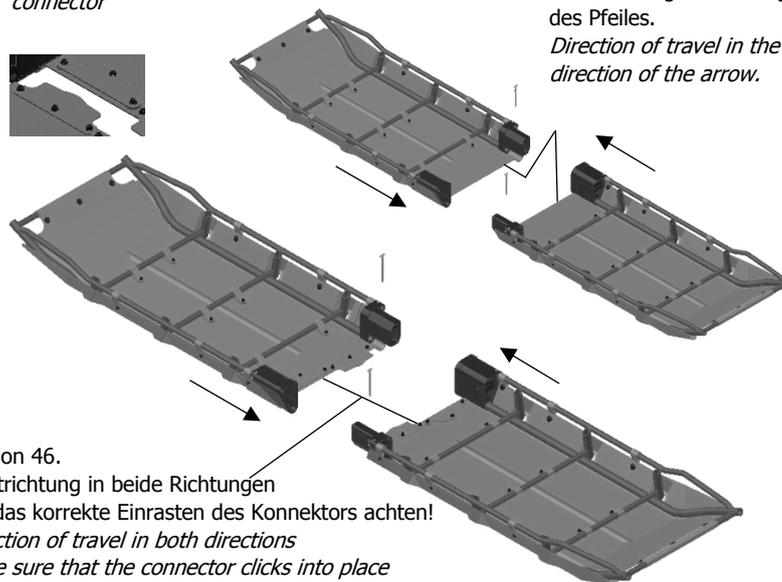Utilization as Back-carrying Device

- Insert the load frame (12) into a load frame positioner (3choices) and secure.
- Set the load on the frame and secure with load securing straps (13).
- Release chest harness (6) and abdominal belt (8). Loosen centre of gravity straps (5) and height-adjustment strap
- Place UT 2000 on back-tighten abdominal belt (8), secure chest harness (6).
- Adjust height-adjustment straps (7). Adjust centre of gravity straps (5) as necessary.

Utilization as Stretcher or Load Carrier (A)

- Couple 2 UT 2000s together (9) (click into place-check by trying to pull it apart!)
- **The two couplings must be secured via the securing pin.**
- Secure the injured person or load into position with the hip straps (8) and the load securing straps (13)
- The stretcher must be carried by at least 2 people.

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Hauling direction! Pay attention to the arrow on the sliding surface.

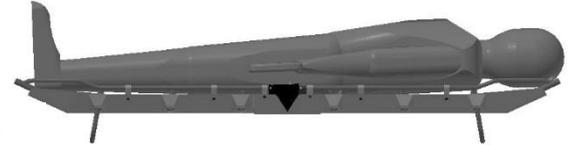
Not Liege

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
Utilisation as Cot or Emergency Bed

- Insert detachable tubular frame (12) as support.

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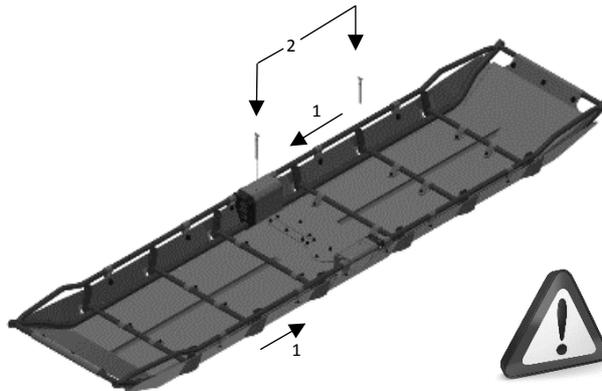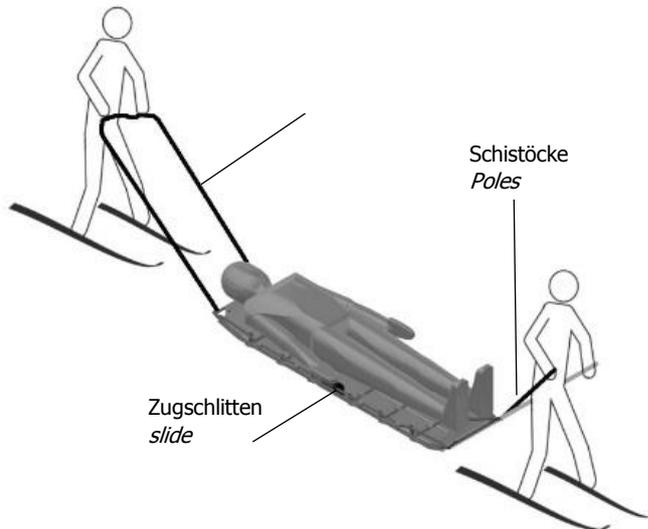
If using a pull able sled, load carrier, first-aid stretcher, the two couplings must be secured via the securing pin.

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Utilisation as Sled in Snow

- Connect the 2 UT 2000 (9) (click into place – check by trying to pull apart!)
- **The two couplings (9) must be secured via the securing pin (9a).**
- Secure the injured or the load into position with the hip straps (8) and load securing strap (13).
- Cross ski poles at the front and secure at the back with cord or straps on the right and left.
- **With the Original UT 2000 (42.) hauling direction to the arrow on the sliding surface.**
- For light loads, only 1 UT 2000 may be used.
- Ackja attachment for UT 2000: see at page 19.

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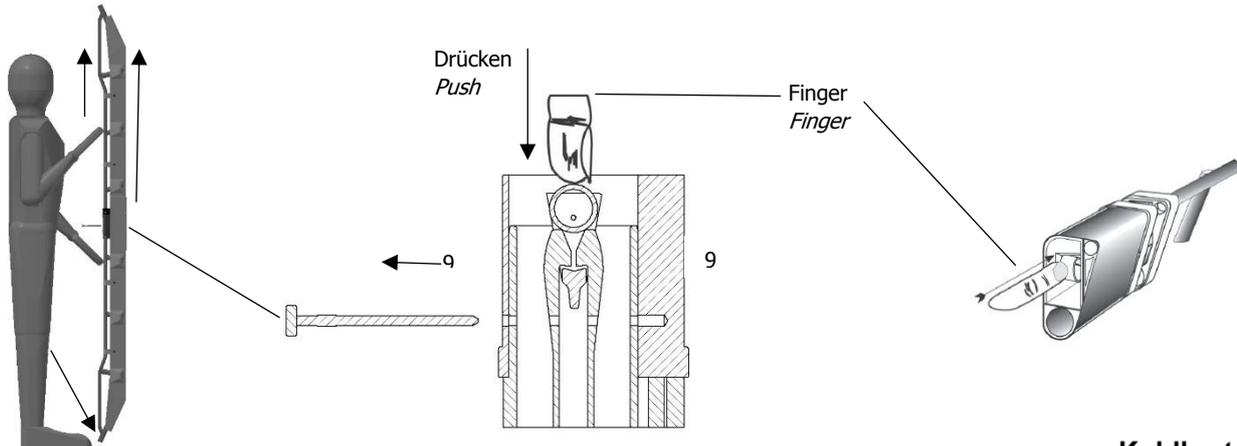
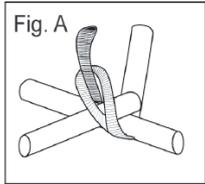
Uncouple 2 UT 2000 Coupling with built-in quick release.

Loosen the two
Coupling locking pins (9a)

Hold the lower UT 2000 with one foot. Pull the upper UT 2000 slightly
upward with one hand. Press with the other hand the quick opening
device build.in in the coupling and pull the device upward. (See figures)

Verdrehte Schlaufe am unteren Rohrrahmen befestigen
 Fix twisted loop to the lower side of the tubular frame.

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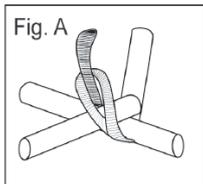
Helicopter and wall recovery

- Two UT2000 are put together in such a way that the couplings can be easily pushed into one another. Pay attention to "click". AT 42. **EIGHTS in the direction of travel!** After engaging and testing the coupling, insert the locking pin (9a).
- Take the hanger straps and mount the hanger depending on the type (pages 10, 11, 12).
- Check all parts for damage, e.g. Cracks, defective seams, corrosive substances, acids, fuel, traces of knots or bruises. Replace the parts if necessary.

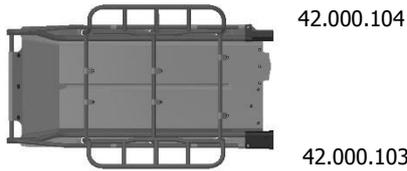
Attention

- People or loads are secured with hip belts (8) and load fastening belts (13). The arms must be fixed when transporting people!

CAUTION

- **The safety regulations for the transport of helicopters must be strictly observed! (see appendix to flight manual 42.030.030 – page 22)**
- Wounding / weaning of an injured person must always be carried out with an anti-rotation rope with a predetermined breaking point 50kp, a predetermined breaking cord.

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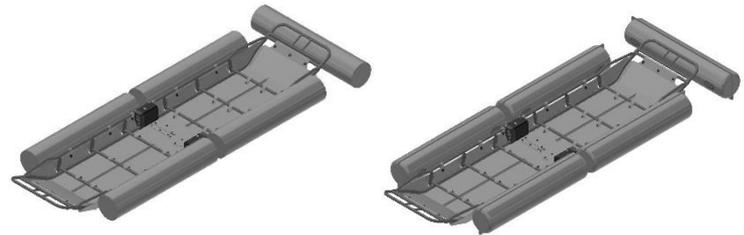
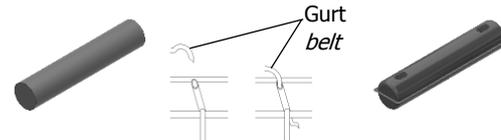
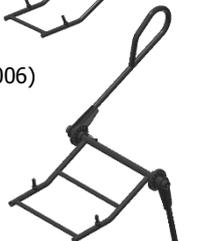
Extension handle for wheel- and Ackja attachment. (45.100.006)

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Movable spar for bike and ackja systems. (46.100.006)
For easier transport or storage with the wheel system.

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

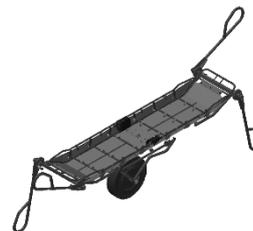
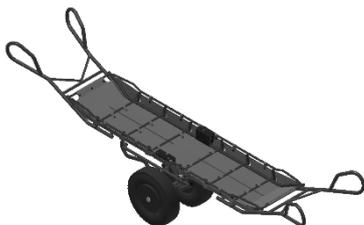
The Ackja systems (45.100.030) 1 man system and (45.100.013) 2 men Ackja system can be used in rough terrain or in snow.

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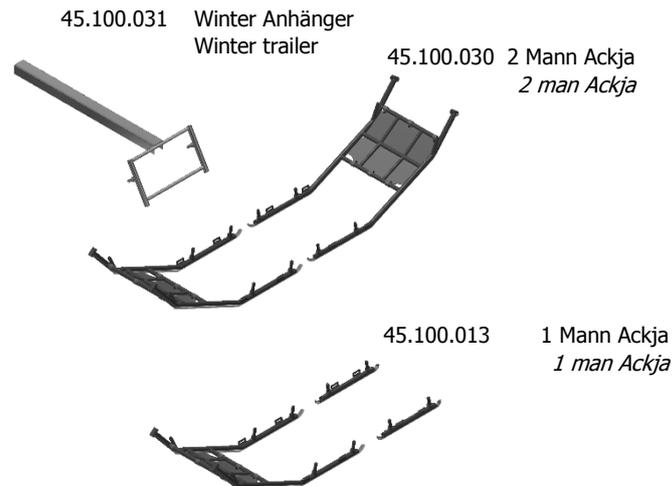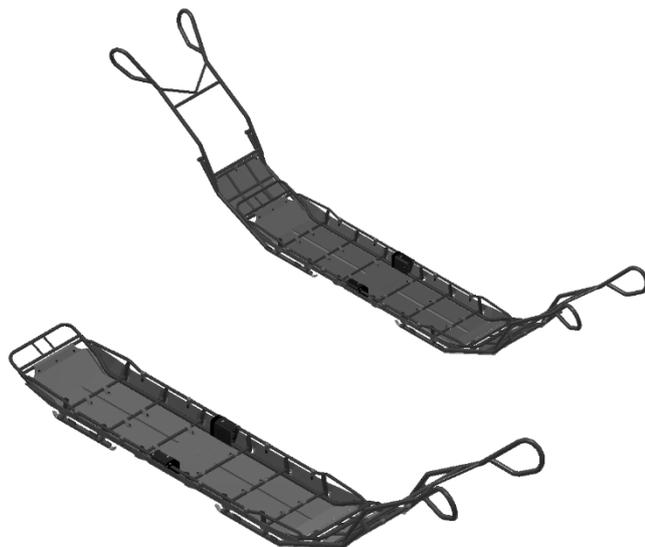*Winter trailer as connection between UT 2000 with Ackjasystem and a Quard / Skidoo. (45.100.031)
An Ackja system is required for this. (45.100.013 or 45.100.030)*

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Head protection / trickle protection

So that the face and eyes of the injured are protected.

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Pad for stretcher with a length of 200cm

Pad for stretcher with a length of 220cm

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Checks for Operational Readiness and Condition

The operational readiness check of the UT 2000 includes a visual examination for damage and a control to ensure that the device is complete and intact.

Parts to be checked:

- Device tubular frame and detachable frames for cracks, breaks and corrosion.
- Insertion possibilities of the detachable frame. (12)
- Coupling function. (9)
- Sliding surface (1) and sliding surface joints (1) for distortion, cracks or breaks.
- Correct seating of all rivet joints.
- Date of expiration of the suspension straps.
- All strap edges for cut, melted and worn spots.
- Seams of the suspension straps for intactness.
- Straps for spots which show damage of the fabric or other damages (corrosive materials, acid, adhesives, fuels, traces of burls or compressed parts).
- Completeness of the strap material.
- Legibility of the operating instructions plate. (2)
- Recognizability of the directional arrows. (9)

Regular Servicing

- Maintenance and Cleaning after each use.
- Check for operational readiness and condition before each use as well as at annual intervals.

Maintenance and Cleaning.

A damp device must be dried and well aired. Soiled straps should be brushed clean with clear water or a mild detergent. Do not use any other chemical cleaners. If the device has been in contact with salt water it should be cleaned immediately with clear fresh water.

Attention: The device should not be allowed to dry in direct sunlight or close to a direct heat source.

- Corrosion traces on the rivet joints shall be removed with a polishing linen cloth and then lubricated with weapon oil.

Storage: UT 2000s are to be stored in dry rooms, away from direct sunlight and protected from the weather, so that the strap material is kept ready for use at all times.

Attention: The UT 2000 may only be stored in a dry condition!

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Maximum service life of the UT2000 / accessories

15 years, from the date of first use (excluding slings, straps)

Regular checks:

• Visual and functional check before and after each use by the User.

Periodic tests:

• Every 12 months from the month of first use.

Condition assessment according to the test card by a qualified / trained person.

Maximum service life of slings and lanyards:

• 10 years from the month of manufacture

Useful life:

The maximum service life mentioned is reduced by damaging influences such as temperatures, mechanical stress from regular and possibly intensive use or exposure to a fall, damage from abrasion, Cuts, chemicals, fusion due to high temperatures, etc.

These influences can have different effects in the reduction depending on the intensity, duration of exposure and combination of different influences.

Intensive use and / or extreme operating conditions such as sharp edges, chemical influences, etc. lead to a reduced rate for safety reasons

Duration of use - The employer must ensure this in the risk analysis of the workplace.

If the sliding surface is too damaged, it can be replaced. Please contact your dealer for this.

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The test card is with the periodic annual review by the to be filled in completely by an expert. This test card makes no claim the completeness of the test criteria and does not bind the expert from his decision on the Overall condition.

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Rotorcraft Flight Manual Supplement, Art. No. 42.030.030 for Personal Rescue Litter UT2000 and Helicopter Rescue Sack BRS

This Supplement shall be attached to the pertinent Rotorcraft Flight Manual when personal rescue is intended with this equipment

Statement-Aeronautical Compatibility

WTD 61 – type testing of aviation equipment for the German Army

UT-2000: Reg.-Nr.: 6545-2005-01 of July 31, 2014

Issue: 5

BRS: Reg.-Nr.: 6545-2005-02 of July 31, 2014

Issue: 4

1. General

The personal rescue litter UT2000-Dual consists of two parts of the UT2000 which are connected together with quick locking devices as described in the appropriate equipment manual. The litter is suspended to the helicopter by its belts, a swivel and a certificated recovery tow to the cargo hook or direct on the hoist. The transport must be assisted by an extern crew member, also hooked to the helicopter.

2. Limitations

Max. speed with empty litter: hovering only

NEW: Maneuvering of winch with BRT/BRS is only permitted when hovering. (slow forward flight for load stabilisation is allowed)

NEW: During rescue of casualty with winch the anti-rotation rope shall be used.

Max. weight of payload: 160 kg

The electric latch disconnect and the cable cutter must be inactive

3. Normal procedure

The patient will be carried in the litter and the transport must be assisted by a crew member.

A full description and maintenance Instructions are given in the pertinent equipment manual.

4. Emergency

In case of an emergency the empty litter can be removed from the helicopter by disconnecting with the manual release.

In case of emergency with an occupied litter the litter and the external attached crew member should be dropped from the helicopter to ground with greatest possible care by manually releasing the connection.

Kohlbrat & Bunz GmbH
A-5550 Radstadt, Austria, Europe
phone: +43 (0) 6452-7193 0, fax +43 (0) 6452-7193 51
email: office@kohlbrat-bunz.com
www.kohlbrat-bunz.com

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